

# Home Care Technical Specification


## Fold Down Shower Seat

# DN7110

### Stock Numbers:

- DN7110 – Fold down Shower Seat
- DN7110OWB – Fold Down Shower Seat, Old World Bronze

### Dimensions:

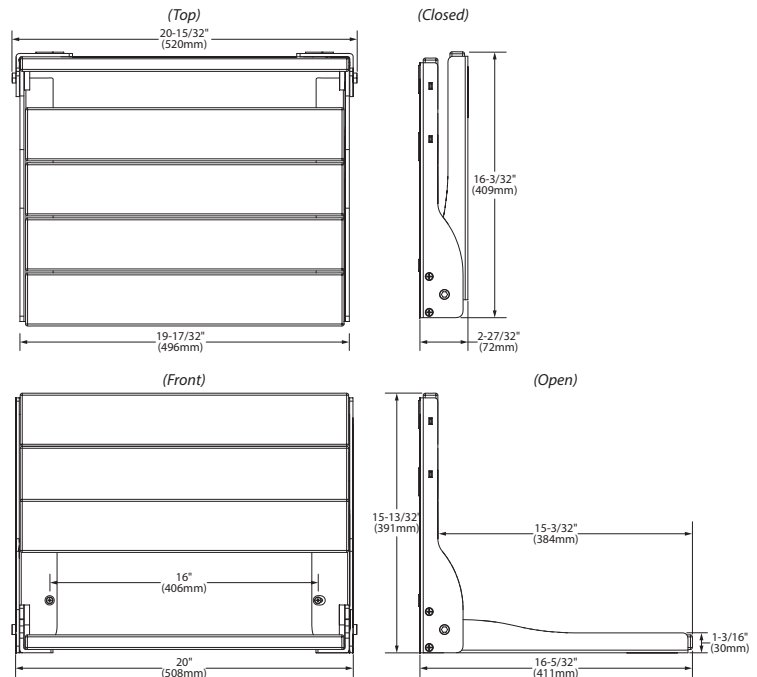

### Materials:

Fold Down Shower Seat is made of Aluminum and Teak Wood

#### CARE:

Clean the Teak seat with a warm, damp cloth. The natural properties of Teak allow for the wood to weather over time in the environment. If you desire to keep the current look of the wood, treat the teak with a certified Teak oil.

**ASSEMBLY TESTED TO  
CONFIRM ADA  
COMPLIANCE**



#### WARNING:

TO PREVENT SERIOUS OR FATAL INJURY:  
Insure product is assembled and used properly.  
FOR USE ONLY BY PERSONS WEIGHING LESS THAN  
400 POUNDS. Failure to follow correct assembly  
procedures can cause the seat to become unstable.  
DO NOT jump on, stand on or tilt. Contact Moen  
immediately at 1-800-882-0116 if any components  
are missing or damaged. Do not modify or substitute  
any components.

FOR QUESTIONS OR ASSISTANCE WITH ASSEMBLY  
CONTACT Moen AT 1-800-882-0116. Consult  
healthcare professional for advice and instructions  
specific to user's needs before use.

### Installation Instructions:

1. Remove the top two back rest panels to reveal the top four mounting holes.
2. Identify the mounting location of the shower seat making sure the shower seat is level.
3. Mark the six mounting hold locations once the desired location is determined.
4. Set aside the shower seat and drill 1/8" pilot hole through tile in each mounting location. Drill a 1/4" hole through the tile only at the same locations to allow the wood screw to pass through.
5. When complete, ensure the wall is clear of any debris.
6. Secure the shower seat to the wall, using the provided six wood mounting screws. Start with the top two mounting hole locations.
7. Once all screws have been installed and tightened, slide the back panels into place.
8. Installation is complete – Test the seat to make sure it is secure.

**Stock Number:** \_\_\_\_\_ Shower Seat  
By Moen Incorporated

25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070  
CORPORATE HEADQUARTERS: (800) 321-8809 • FAX: (800) 848-6636  
INTERNATIONAL: (440) 962-2000 • FAX: (440) 962-2726  
CUSTOMER SERVICE (USA): 800-882-0116 • FAX: (888) 379-2720  
WWW.MOEN.COM  
©Copyright Moen Incorporated

# MOEN®